SmartCase SC1530 series

Manual


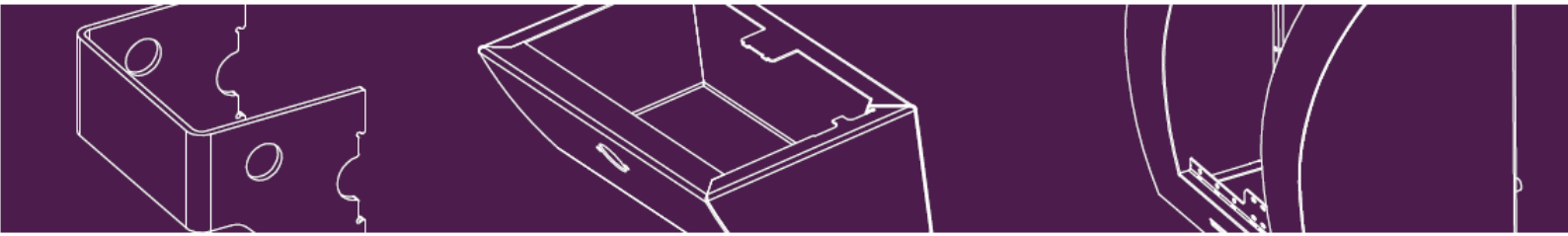
Table of Contents

WHAT IS A SMARTCASE?	3
FOR YOUR SAFETY	3
WARRANTY	4
PRODUCT OVERVIEW	5
INSTALLATION	6
OPERATION	10
MAINTENANCE AND TROUBLESHOOTING	10
END OF LIFE	10
TROUBLESHOOTING:	11
GLOSSARY	12
APPENDIX A: TECHNICAL SPECIFICATION	13

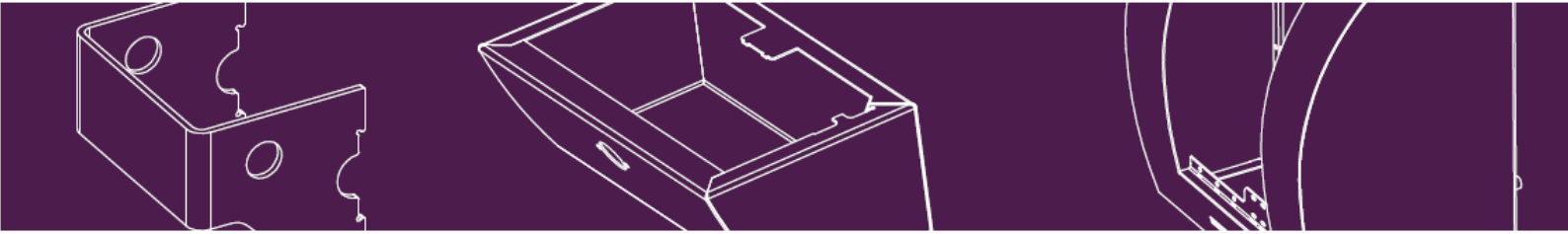What is a SmartCase?

The AED (Automatic External Defibrillator) is an important device for primary help in case of a cardiac arrest. It is of a vital importance that the AED is protected, secured and available 24/7 for Immediate assistance in case of an emergency. Your SmartCase cabinet provides the ultimate protection and assures day and night availability of the AED.


The SmartCase outdoor cabinet is designed to protect the AED against external weather conditions. The stainless steel box combined with a polycarbonate window provides the ultimate protection against corrosion and impacts and offers a clear sight of the AED. To maintain a temperature above 0°C, the outdoor SmartCase is equipped with a temperature sensor and electrical heater. The cabinet is insulated to save energy. Additional protection against high summer temperatures is achieved by active ventilation.

For your safety

Read the user manual carefully. All operators should review these safety considerations before placing the SmartCase AED Cabinet into service. Not following the instructions may result in dangerous situations. Follow all recommended maintenance instructions. If a problem occurs, disconnect the cabinet from the powergrid and contact the technical service. Do not use the device until it has been inspected by qualified personnel.


		Symbol
Warning:	Warning statements alert you to conditions or actions that can result in personal injury or death.	

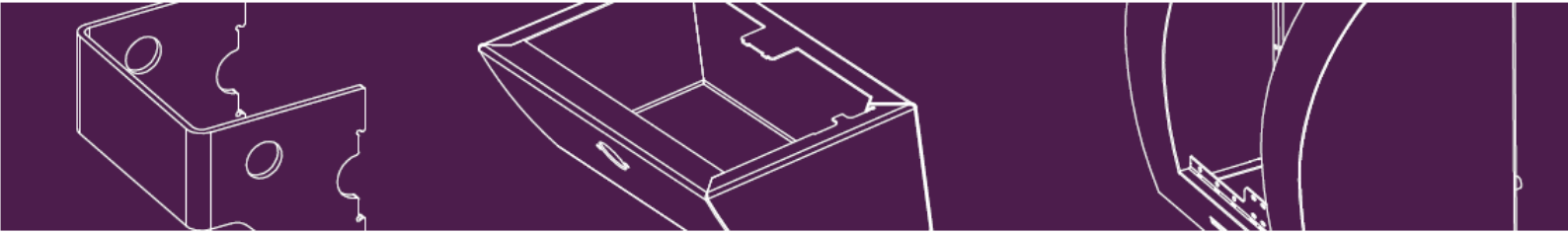Important: GT MEDICARE is neither responsible for the contents of this document nor for the loss of data, income, direct or indirect caused damage. GT MEDICARE preserves the right to adapt and to improve this document and all related products in this user manual, without preceding notification. The contents of this document are provided without any form of warranty.

Warranty

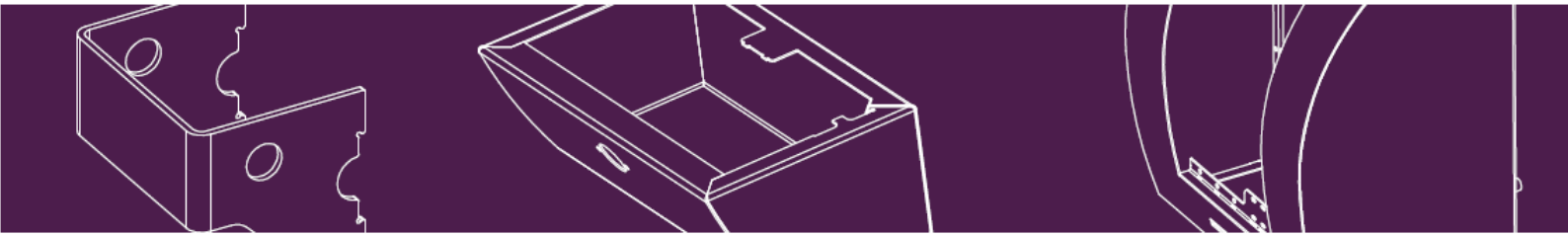1. GT MEDICARE is responsible for the quality of services and for the soundness and good quality of the products supplied to the best of its knowledge, with the provision that the guarantee does not exceed the following provisions mentioned.
2. The warrentee is based on send and return. If this service is used, the purchaser must take care of the correct packing of the product such that damages during transport are avoided. If the defects are caused by purchaser the repair and shipping costs will be charged to purchaser.
3. The purchaser is not entitled to warranty:
 - A: If the buyer has neglected the products.
 - B: If the buyer has made changes to the products or repairs which were not performed by or on behalf of GT MEDICARE.
 - C: In case of improper or careless use, incorrect connection, wrong voltage, lightning, damage due to moisture or other external products or misfortunes.
 - D: If the device is not appropriately maintained.
 - E: If the device is used or installed with non original or inferior accessories.
 - F: If the buyer dealt with the goods with negligence.
4. GT MEDICARE warrants that the Products do not contain serious defects at the time of delivery.
5. Reseller will report defects, damages or non-conformity of the Products, which upon inspection can reasonably be determined, in writing to GT MEDICARE within 10 days after reception. Reseller will report all other defects in writing to GT MEDICARE within one month from the date of delivery. Products, which suggest that they are non-conforming or damaged, should be returned to GT MEDICARE immediately for inspection by GT MEDICARE. Claims, which were not reported according to the terms in this paragraph, will not be accepted. GT MEDICARE will, in case of responsibility for the non-conformity or damage to the product, at its option, repair the Products, replace them with new ones or refund the purchase price of the Products. GT MEDICARE will have no other obligation or liability towards Reseller with respect to non-conforming or damaged products other than in the previous sentence displayed.
6. The duration of the warranty is 24 months. Scratches and damages to the cabinet and wear & tear are outside this provision. The producer is only obliged to fulfil the obligations arising from this warranty. The producer is not liable for indirect or consequential damages. In all cases, the manufacturer's liability is limited to the selling price of the products sold.
7. Where a claim can be attributed to a modification or improper use of the product, the guarantee is under all circumstances void.
8. The recipient will investigate all received equipment immediately. The customer must inform GT MEDICARE in writing within ten (10) days after receipt of any defects in the consignment. If the deficiencies are not reported within a period of ten (10) days, the shipment was considered to be complete. Claims are only possible within the warranty period. If a complaint is accepted by GT MEDICARE, then GT MEDICARE is only obliged to replace the defective goods or repair, without the customer being entitled to claim any compensation whatsoever. Submitting a claim does not relieve the customer from payment.

Product Overview

- SmartCase AED Wall Cabinet, with pre-installed:
 - On Uppertray:
 - 24V Adapter
 - Cable for 230VAC connection
 - LED lights (red and blue)
 - Alarm, +/-95dB
 - Inside Cabinet
 - fan for active ventilation
 - electrical heater & AED bracket
- Quick Installation guides (2x)
- Installation materials
- Doorknob installation kit

Installation materials:

Washers (4x)

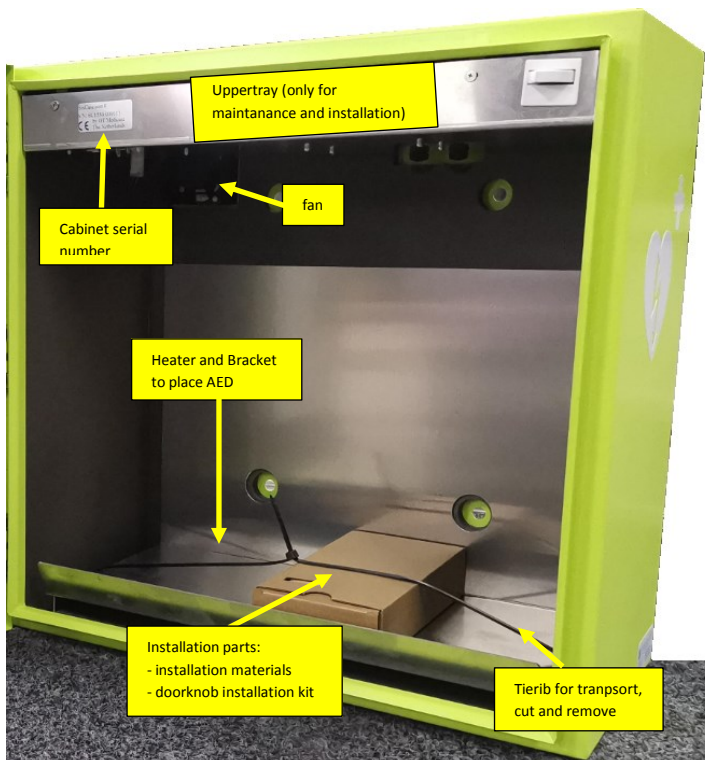
M8 Nuts (4x)

Nylon spacers (4x)

Wall plug (4x)

Dowel screw (4x)

Inside view of cabinet

Doorknob installation kit:

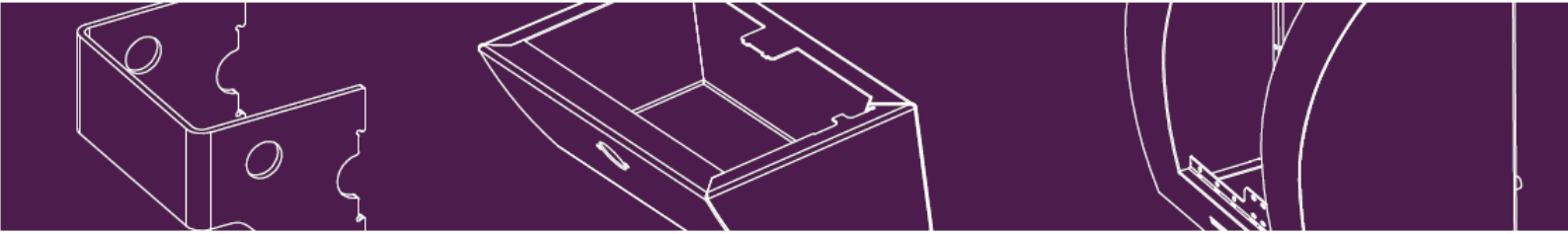
Turn knob (1x)

Rubber ring (1x)

Nut (1x)

Cam lock (1x)

Screw (1x)

Installation

Location for the AED cabinet:

- It is advised to choose a location which has easy access and is visible.
- The intended location for the cabinet must have a flat surface and sufficient clearance surrounding the cabinet (advice is >1 m clearance) to allow proper functioning and installation. There should be no obstacles underneath the cabinet.
- The installation tools supplied are intended for mounting exclusively on flat surfaces of wood, brick or concrete. Panels/wall must have a minimum thickness of 3mm. Always use the supplied spacers.
- We recommend installing the cabinet in the shadow to prevent high temperatures inside the cabinet.
- The location must have access to the powergrid.

Mounting on brick or concrete:

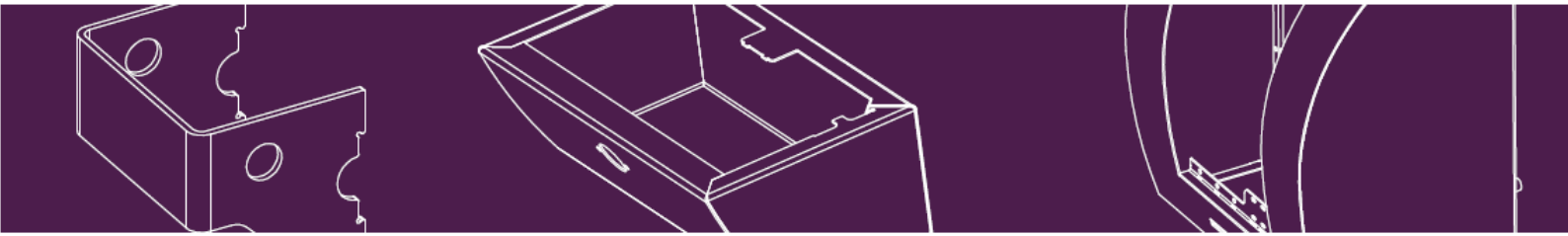
Drill 4 holes for the attachment bolts (dowel screws) with a spacing as shown in "Cabinet installation instructions". Drill the round 10 mm holes to a minimum depth of 60 mm, to ensure there is sufficient depth for the plug. We recommend drilling the upper holes at between 1.5 and 1.6 metres from the ground. The holes should always be drilled in the middle of the brick. Insert the M8 dowel screws in the plugs until they protrude 20mm from the wall and slide the nylon spacers over the dowel screws. Drill the 12 mm feeder hole (for the power grid connection) as shown in "Cabinet installation instructions". Before mounting the cabinet on the wall, first install the feeder cable.

Be careful: If the wall is not very strong, we recommend to use Fischer VS 100P. After spraying the fluid into the holes, insert only the screws – do not use the plugs again. Read the instructions on the Fischer label carefully and make sure that you clean the holes thoroughly after drilling.

Mounting on wood:

Drill small holes of approximately 4 mm diameter in the wood and screw the wiretap firmly into the wood and then mount the SmartCase as instructed above. We recommend drilling the upper holes at a height of between 1.5 and 1.6 m from the ground.

Follow the installation instructions as provided on the next pages.

Cabinet installation instructions:

Installation materials:

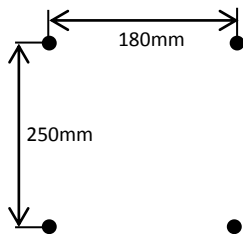
- Washers (4x)
- M8 Nuts (4x)
- Nylon spacers (4x)
- Wall plug 10mm (4x)
- Dowel screw (4x)

Suitable for installation on brick/concrete or wood


IMPORTANT:

- Minimum required wall thickness: 3mm
- Install on a flat surface
- Always use the spacers
- Recommended height: 1.5 – 1.6m upper holes (above ground)
- Install in shadow
- For electrical power connection, see next page

Installation

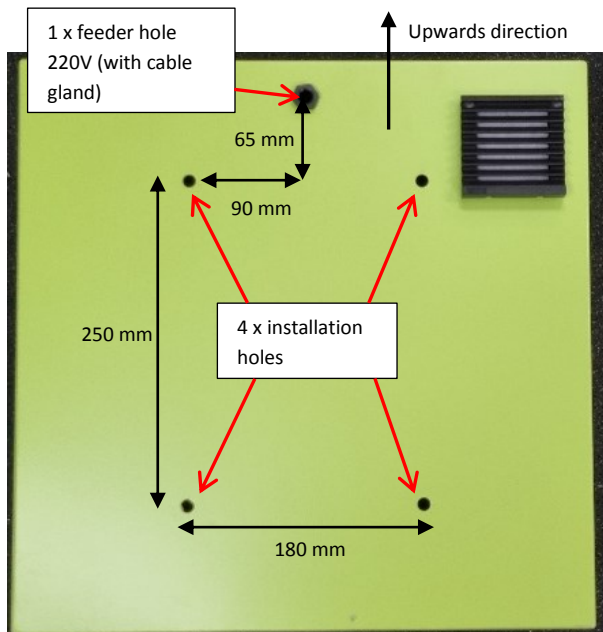- 1: Drill 10mm holes (4x)
- 2: Insert Plugs and dowel screws (20mm protrusion)
- 3: Place spacers
- 4: Feeder hole: Drill a 12 mm hole through the wall for electric connection to 220/230V AC.

5: Install cabinet and place washers and M8 Nuts on the inside

Electrical connection of the cabinet: to be carried out by qualified personnel only.

1) Install the cabinet on the wall (see previous steps). Open the door and open the upper tray by removing two screws: 1 x front upper left hand-side and 1x front upper right-hand side.

Disconnect the power for the cable (220V/230V AC) going to the cabinet location!

2) Install electrical cable (220V/230V AC, not part of the installation kit) and feed it through the feeder hole at the back of the cabinet (use the cable gland to secure the cable).

3) Disconnect the male connection (a) from the female connection (b). Unscrew to open the male connector (a) and connect the 220V/230V electrical cable to the male connection: ensure the correct connection of the wires, where the ground is to be applied to middle male connector. When finished, close the male connector (a) with the screws & push the male connector (a) back into the female connector (b). Check if the connection (d) to the adaptor is pushed in.

Double check that all 220V/230V AC wire connections are inserted correctly, that they are properly secured and that there are no open connections.

4) Make sure that local the earth cable (c) is still connected to the cabinet in the back.

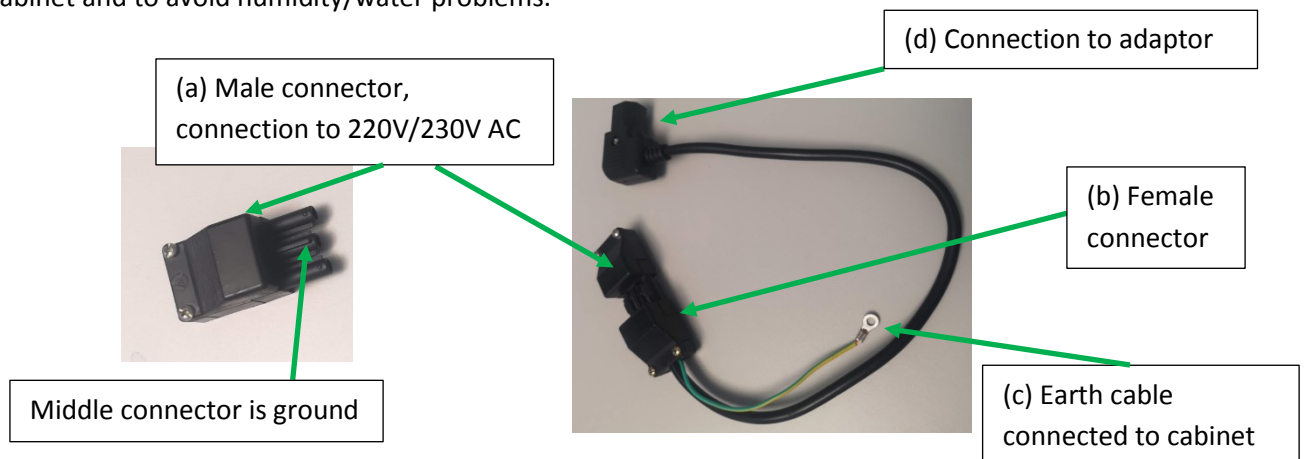
5) Close the upper tray and lock with the two screws from step 1). Make sure that there are no loose 220V/230V AC cable(s) coming outside the tray to ensure the safety of the users. Local cables of the cabinet are present for heating and fan cooling and are safe (24V/DC).

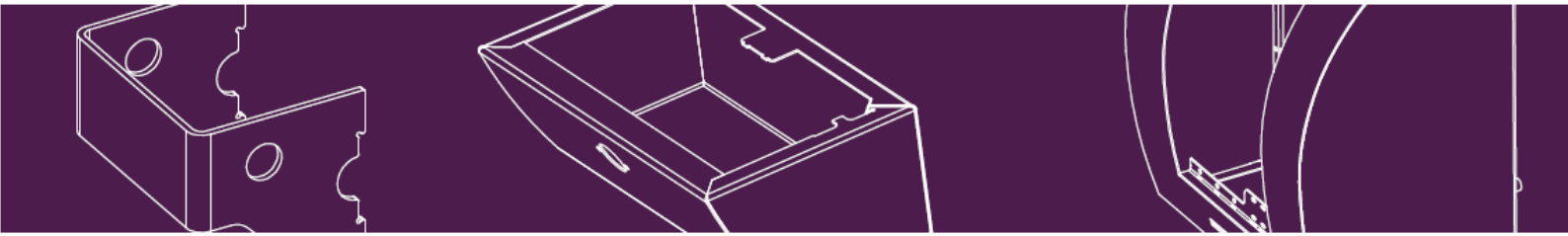
6) Close the cabinet door and connect the electrical cable to the power grid (220V/230V AC).

7) Check the cabinet: a blue light should be on inside the cabinet. When the cabinet is opened, a red blinking light should go off with an alarm sound.



Always keep the door of the cabinet closed (also after usage), to ensure the correct heating of the cabinet and to avoid humidity/water problems.

INSTALLATION OF THE TURN KNOB

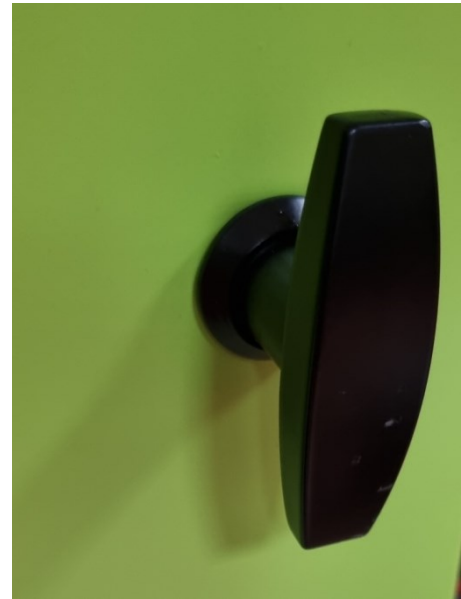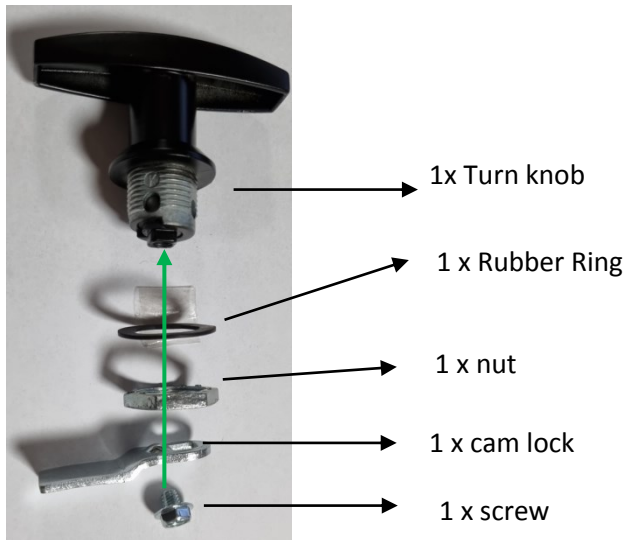Installation materials:

Installation

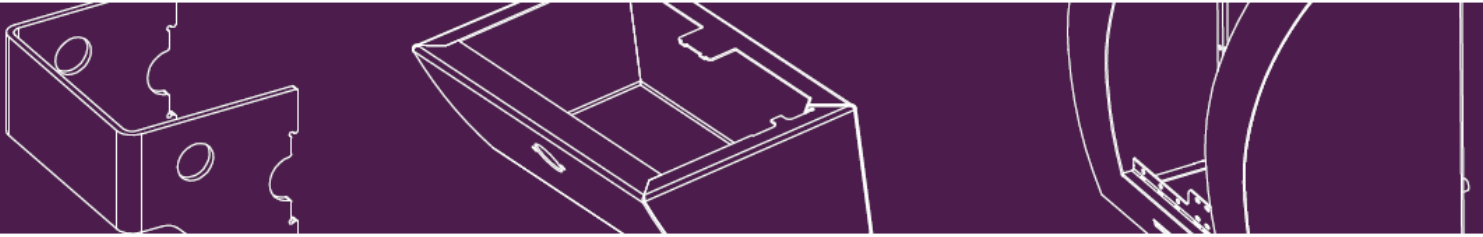
1: Place rubber ring on turn knob and fit through the hole in door.
Note: you can change the locking/opening by inserting the turn knob in different angles with steps of 90 degrees.

2: apply nut and tighten. Then apply camlock and finally screw. Note that camlock can only fit in one position on turn knob

3. It is advised to follow the opening direction as indicated on the sticker on the outside of the cabinet. Adjustment of turn knob opening is performed as follows: uninstall all parts and rotate the turn knob in the desired new position for correct opening/closing and redo the installation of step 1 and step 2.

Operation

The opening and closing instructions in case of emergency are shown by the sticker on the door.
NOTE: The cabinet must always be closed and locked to ensure its functioning.

Opening of the cabinet:

Turn the door handle to open the door of the cabinet. When the door is open the alarm will be triggered with intermitting sound and blinking red lights (alarm automatically turns off after approx 30 seconds).

Closing of the cabinet:

Close the door (turn handle 90 degrees if closing is blocked). If the alarm is sounding, it will go silent. Turn the handle 90 degrees to lock the cabinet.

Maintenance and Troubleshooting

It is advised to check the wall cabinet **monthly**. Checking should consist of a visual check if damages are present which could affect the operation of the cabinet and a check of the alarm (by opening and closing the door). While doing so, you should take careful note of the status of the defibrillator (please check manual of AED).

The uppertray should only be accessed by qualified personnel (high voltage).

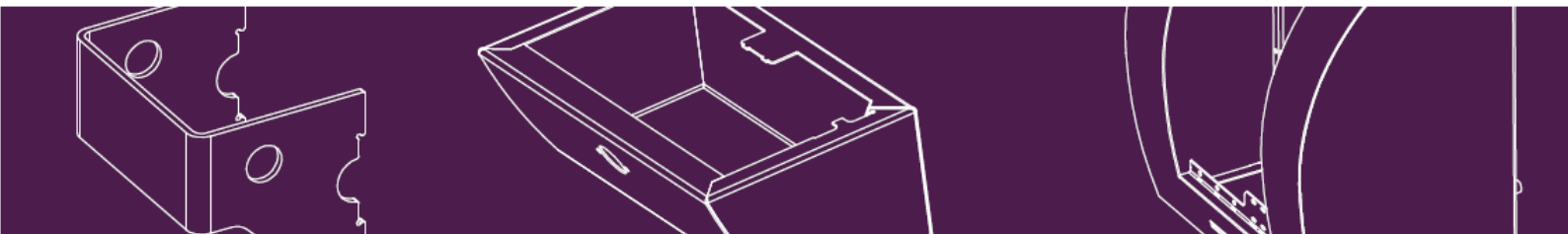
Upon discovering a defect, please contact your dealer of SmartCase. Depending on the type of defect it may be advisable to remove the AED and inform the users about the status (unserviceable).

Temperature: If you have any doubt about the functioning of the electrical heater (outside temperature lower than 0°C / 32°F), please remove the defibrillator and contact your dealer of SmartCase. The cabinet are equipped with an active ventilation system. Around 30°C the fan will be switched on to release the heat from the upper part and taking in cold air from the bottom. Nevertheless, we recommend installing the cabinet in the shadow to prevent high temperatures (above 50°C).

End of life


When the cabinet is taken out of service permanently, make sure to dispose the cabinet correctly as it contains electronical parts.

Troubleshooting:

Symptom	Action
LED Lights are out or alarm is not working	<p>Open the upper tray (only by qualified personnel):</p> <ul style="list-style-type: none"> - Warning high voltage (220/230V AC) present.  - Check the power supply and adapter: the LED on the adapter should light up continuously (a flashing LED or no light indicates a malfunction) - Make sure that all connectors are properly connected <p>If the unit continues to fail, take it out of service and contact the dealer of SmartCase.</p>
Temperature (heat) in cabinet not working or malfunctioning	<p>The temperature is controlled using temperature sensors in the cabinet. Heating turns on if the outside temperature is below 0°C and turns off if the inside temperature of the cabinet reaches 20°C. Note: for correct functioning of the heater, the cabinet door must be closed.</p> <p>When it is suspected that the heater is not working:</p> <ul style="list-style-type: none"> - Open the door. The alarm should sound. If the alarm is not working follow the steps: “LED lights are out or alarm is not working”. - Check the temperature inside the cabinet by touching the tray on which the AED is placed (outside temperature must be < 0°C). The tray should feel warmer compared to the outside temperature. If this is not the case: take the cabinet out of service, remove the AED and contact the dealer of Smart case. <p>Heater is continuously on:</p> <ul style="list-style-type: none"> - remove the AED. - Check if the door of the cabinet is correctly closed. If the door is open, then close the door. Wait for 5 minutes and check if the heater has turned off. - If the unit continues to fail, remove the AED, take it out of service, unplug the cabinet from the power grid and contact the dealer of SmartCase.
Cabinet is not closing correctly	<ul style="list-style-type: none"> - Check if anything is blocking the door. - Check if the lock or door is deformed in the area of the locking mechanism <p>If the unit can't be repaired, take it out of service and contact the dealer of SmartCase.</p>

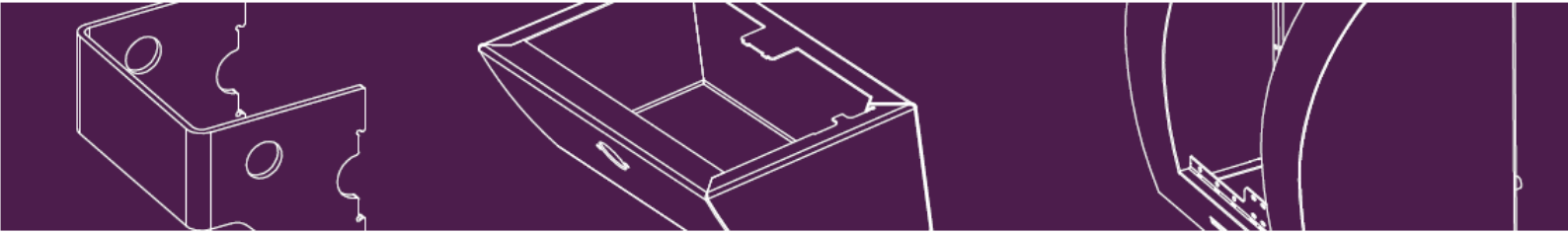Glossary

AED

The AED (Automatic External Defibrillator) is an important device for primary help in case of cardiac arrest. It is vital that the AED is protected, secured and available 24/7 for immediate assistance in case of an emergency.

Operator/ user

The person who is responsible for the cabinet and AED. His/ her duty is to check the status of the AED and Cabinet and assure operation complying to the directives as described in this manual.

Online service

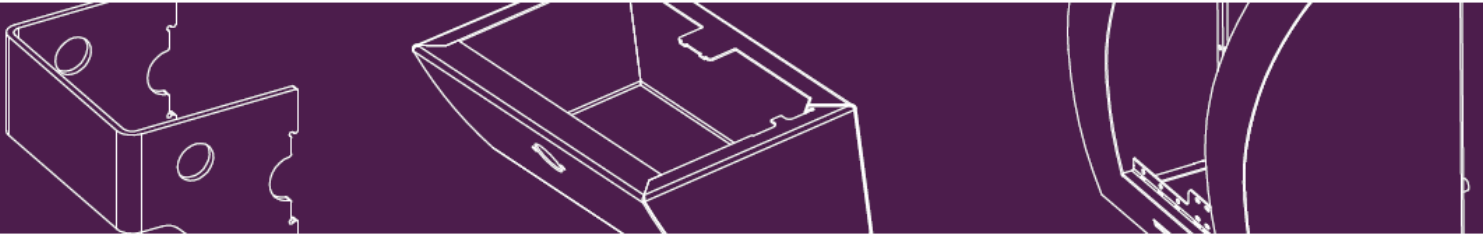
For more information or online help, please visit our website www.sixcase.com.

User Manual: The manual contains all necessary information about operating the SmartCase. Updates are published on our website.

Disclaimer: GT MEDICARE is in no way responsible for the availability or serviceability of the AED stored in the SmartCase cabinet. GT MEDICARE is neither responsible for the installation, use or operation of the cabinet/AED nor for the loss of data, income, direct or indirect nor for caused damage to third parties. Exceptional external conditions, such as low temperatures, defect parts or other external elements may result in unserviceable AED's. GT MEDICARE preserves the right to adapt and to improve this document and all related products in this user manual, without preceding notification.

Contact

GT Medicare BV
Industrieweg 36A
2382 NW Zoeterwoude
The Netherlands
T +031 (0) 71 2100 220
E info@sixcase.com

Appendix A: Technical specification


Technical Specification

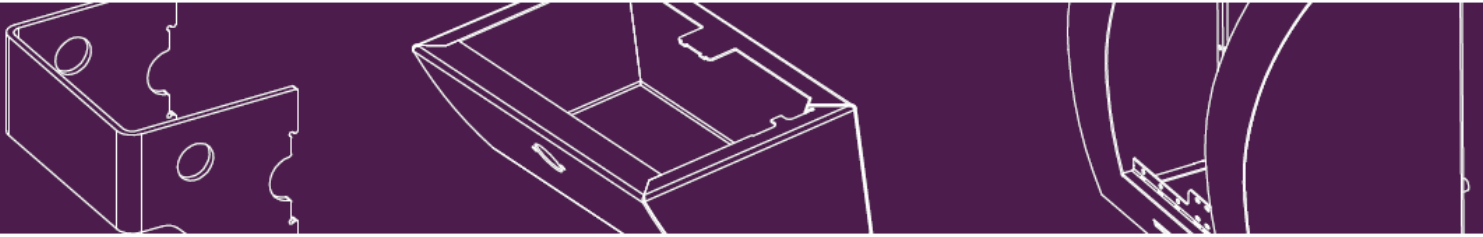Type	SC1530
Weight	11 kg
Dimensions	44(h)x44(w)x24(d)cm
Shockproof	IEC61439-5 clause 10.2.101.2/3
Waterproof	IP54
Cooling	Active ventilation
Heating	Electrical until -20 °C outside temperature
Corrosion resistance	Stainless steel with coating
Window	Impact proof polycarbonate
Installation kit	Included (wood & stone)
Power	24V Adapter (included)
Opening	Door knob (90 degree turn)
Lighting	LED red en blue
Acoustic alarm	+/- 95dB intermittent
Max peak consumption	max 70W (heater continuously on)

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